Hemopropin[®] pomada

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EN)

Hemopropin® ointment

FOR FASING THE DISCOMFORT CAUSED BY THE HEMORRHOIDS

Medical device

Ingredients: PETROLATUM, LANOLIN CERA, PROPOLIS EXTRACT, ANTHEMIS NOBILIS

Intended use: Hemopropin® ointment provides protective film for decreasing the rectal mucosa irritation and painful defecation. Hemopropin® ointment creates protective film on the damaged rectum mucosa and thus facilitates the unpleasant symptoms caused by haemorrhoids (flatulence, pain, burning, itching), which are usually the result of secondary inflammation in the anal area. The resulting film prevents contact between the stool and damaged mucosa. With facilitated defecation (without pain and strain), mucosa recovers very quickly.

Instructions for use: During the acute periods, apply Hemopropin® ointment on clean anorectal area in the morning and in the evening. Later, when symptoms relieve apply only in the evening. It can also be applied by using applicator put on the tube instead of the cap. Before use, remove the protector from the applicator's top. Right after using the applicator, remove it from the tube which should be closed by the cap. Thoroughly clean the applicator with water, dry and close it with the protector. If the ointment is applied with fingers, take care of hand hygiene and apply the ointment with fingertips.

Contraindications: Allergic reaction caused by hypersensitivity to an ingredient of the product.

Possible side effects: There have been no side effects reported while using Hemopropin® ointment. In the case of hypersensitivity to the components of Hemopropin® ointment, it may occur reaction in form of redness, rash and itching of the skin. If you notice any of the symptoms, consult your doctor or pharmacist. If you notice any of the symptoms not described here, consult your doctor or pharmacist.

CAUTION: Persons allergic to propolis should not use this product! Keep out of the reach of children.

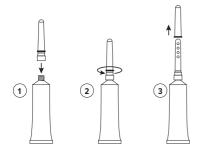
Safekeeping: Store in a dry and cool place. The product can be used for 2 months after opening.

Net quantity: 20 g REF 1101

APIPHARMA d.o.o., Jeronima Kavanjina 26, 10090 Zagreb, Croatia, EU

Batch number and Expiry date are printed on the package.

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